SEATING SYSTEM
SITZSYSTEM
SYSTÈME D'ASSISE
SISTEMA DI SEDUTA
ZITSYSTEEM

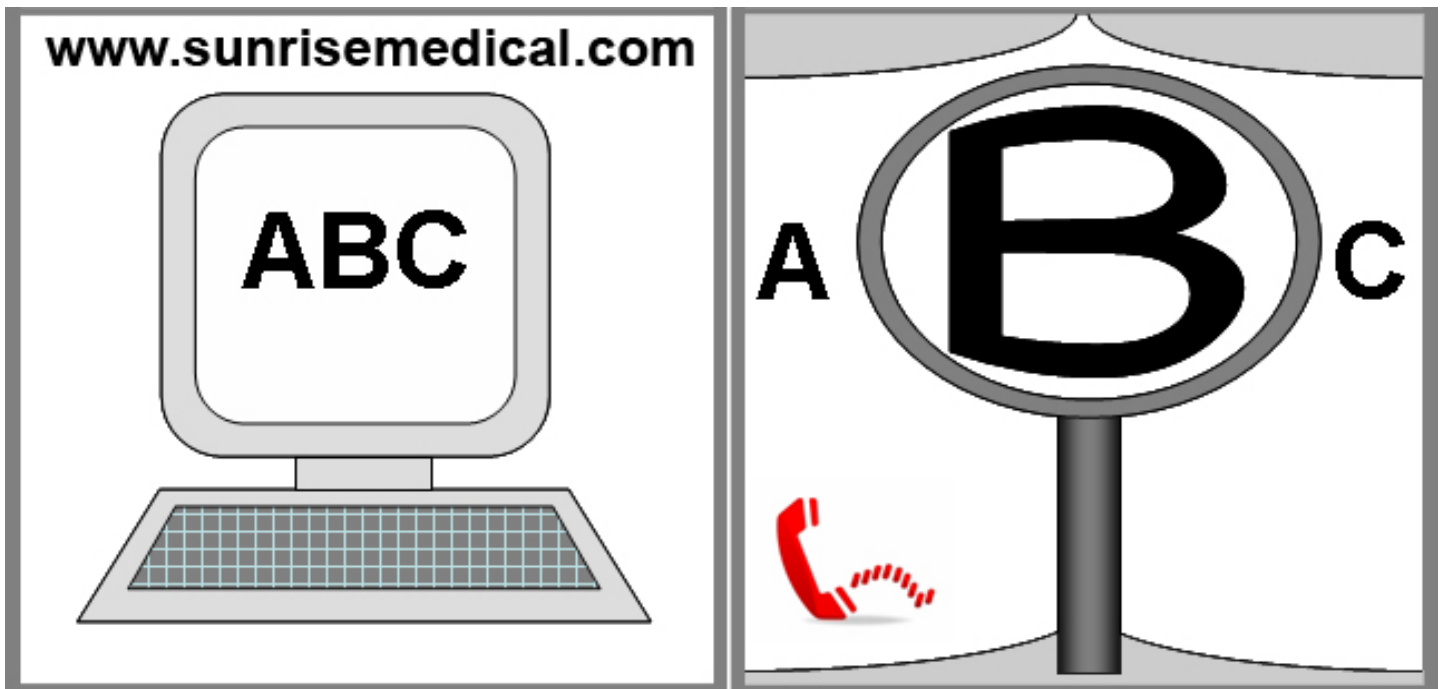
Pluton

DIRECTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
MANUALE D'USO
GEBRUIKERSHANDLEIDING

zippie®

294000.22003

If you are visually impaired, this document can be viewed in PDF format at www.SunriseMedical.co.uk

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
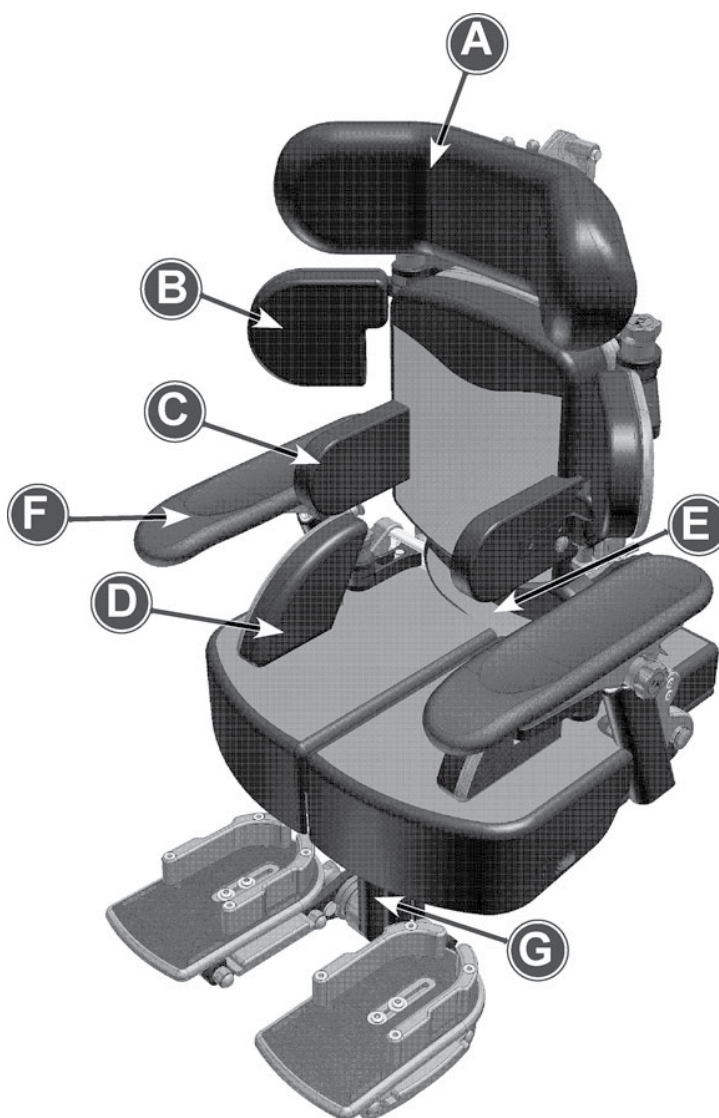
Measurements & Usefull Information

MEASUREMENTS (mm)	Pluton 1		Pluton 2		Pluton 3		Notes
	MIN	MAX	MIN	MAX	MIN	MAX	
A Seat Depth	200	350	300	450	400	550	
B Seat Width (Between Hips)	150	250 (315)	230	330 (395)	310	410 (475)	(mm) - When Hip Pads Removed
C Seat to Top of Shoulder	300	440	390	560	480	680	
D Chest Breadth	150	270	230	350	310	430	
E Seat to Footplate	190	370	260	440	330	510	Leg Length has 2 Settings to Achieve Min/Max
F Seat to Back Angle							
Seat Weight	14.5 kgs		17.0 kgs		19.5 kgs		Guideline (inc Arms, Hips, Lats, Footplates)
Load capacity							Always Refer to Chassis Capacities
Age Guideline	4-9 years		8-12 Years		12-adult		
Measurements when fitted to Multi Height & Tilt Base							
G Seat to Floor			100mm Castor: Min - 355 / Max - 815			100mm is Standard	
H Tilt / Prone Angle							
I Base Footprint							
<p>The Pluton is a modular seating system where at assessment the selection of Seat, Back and Leg Widths & Lengths can be specified. The Seating system will then be built to ensure best fit & maximum growth from the outset. Dimensions provided here provide guidance on the standard sizes available.</p>							

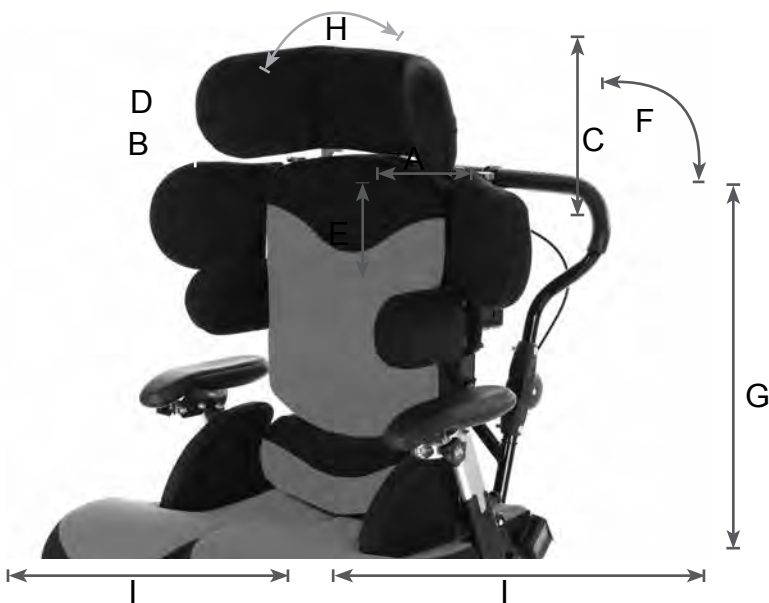Please note that all measurements given are for indication only. Sunrise Medical can tailor the chair to your precise requirements. Our Product Specialists will assess your child to ensure the chair is fit for purpose both now and for the future.

Introduction

Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL.

The Pluton is designed to provide comfort and postural support. The system is available in five sizes to accommodate children from as young as 6 months through to adult.

This instruction manual contains very important information about the Pluton seating system, how to use it safely and obtain the best results from it. Please read all the information contained in this manual before using the chair and retain for future reference. Ensure everyone using the equipment is aware of the contents of this manual and understands how to use the equipment safely.

These instructions provide guidance on the adjustments for professionals, but they also give information on how the equipment should be safely used, maintained, checked, and correctly assembled by anyone who uses the seating system.

In every case the equipment should be supplied via qualified professionals who will have adjusted the equipment, checked its compliance and tested it appropriately.

The seating system should only ever be adjusted by qualified professionals and persons that are suitably trained. For further information please do not hesitate to contact us - contact details are below.

Thank you for purchasing our product, we hope you enjoy your new seating system.

Use

Attendant configurations, are exclusively for a user who is unable to walk or has limited mobility, for their own personal use indoors or outdoors.

The maximum weight limit (includes both the user and any weight of accessories fitted to the wheelchair) is marked on the serial number label, which is affixed to the crossbar or stabiliser bar below the seating system.

Warranty can only be taken on if the product is used under the specified conditions and for the intended purposes.

The intended lifetime of the product is 5 years.

Please **DO NOT** use or fit any 3rd party components to the seating system unless they are officially approved by Sunrise Medical.

DANGER!

- Do not fit any unauthorized electronic equipment.
- Do not fit powered or mechanically operated mobility devices, or any other device that changes the intended use or the structure of the seating system.
- Any combination with other medical devices requires approval directly from Sunrise Medical.

Area of application

The variety of fitting variants as well as the modular design mean that it can be used by those who cannot walk or have limited mobility e.g. because of:

- Paralysis
- Loss of extremity (leg amputation)
- Extremity defect deformity
- Joint contractures/joint injuries
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia.
- When considering provision, please also note the body size, weight, physical and psychological constitution, the age of the child, living conditions and environment.

Operating Conditions

The seating system can be used indoors as well as outdoors in attendant configurations, on dry, firm, relatively smooth surfaces. It is not recommended for use in a storm, torrential rain, snow and ice or other extreme weather conditions.

Contents

Measurements & Usefull Information	4	18.0 Lateral Support Adjustment	23
Introduction	5	Height & Width of Supports	23
Use	5	Flip Away Action (Where Fitted)	23
Area of application	5	19.0 Arm Rest Adjustment	24
Contents	6	Adjusting Arm Height	24
Definitions	7	Adjusting Arm Angle	24
Definitions of words used in this manual	7	Removing The Armrest	24
1.0 Important Safety Advice	8	20.0 Legrest Adjustment/Removal	25
2.0 Warranty	9	Split Option:	25
3.0 Brakes	10	Angle Adjustment	25
Operating Brakes, (Indoor Base Frames)	10	Removing the Leg Stem	25
Applying Brakes	10	21.0 Foot Position Adjustment	26
Releasing Brakes	10	Footplate Angle Adjustment	26
4.0 Chest Harness-Adjustments	10	Flipping Up the Footplate	26
Chest Harness	10	Footplate Height	26
Fixing Onto Chair	11	Foot Holder Positioning	26
Fastening of Belts	11	Foot Size	26
5.0 Lap Strap Safety	11	Fore and Aft, Rotation	27
Lap Strap Fitting	11	Side to Side/Width Adjustment	27
Harness and Lap Strap Positioning	12	Foot Straps	27
6.0 Adjustment Points and Lever Use	13	22.0 Dynamic Back Set Up	28
Adjustment Points Underneath Seat, (Fig.6.1)	13	Setting the Strength of Resistance & Power of Return	28
Effective Operation of Lock Levers (Where Fitted)	13	Increasing the Air Pressure	28
7.0 Lever Use-Using Index Plungers	14	Decreasing the Air Pressure	28
Using The Index Plungers (locking Variety Only)	14	Setting The Power of Return	29
8.0 Adjusting Shoulder Protector Pads	14	Disengaging the Mechanism	29
Pad/ Shoulder Width	14	Maintenance Schedule	29
Pad/Protraction Angle	14	23.0 Cleaning and Care	30
9.0 Seat Unit	15	Fabric Upholstery Care	30
Mounting	15	Metal Framework	30
Removal	16	24.0 General Maintenance	30
Manual Seat Height Adjustment	16	25.0 Servicing by Approved Dealer	32
Powered Base Height Adjustment	16	What should be completed during a service?	32
10.0 Tilt In Space	17	What happens if a fault is found?	32
Multi Height & Tilt Base (Fig.10.1-10.3)	17	Who should carry out the service?	32
11.0 Folding Push Handle	17	Frequency of Service	32
Multi Height and Tilt Base (Fig.11.1-11.2)	17	How To Book A Service	32
12.0 5-Star Base Adjustment	18	26.0 Service Record Log	33
To Raise The Chair	18	28.0 Inspecting and Reissuing of Equipment	34
To Lower The Chair	18	Cleaning	34
To Perform the Tilt in Space on the 5 Star Base	18	Assessment of Fabrics / Foams	34
13.0 Back Rest Adjustment	19	Electrical and Electronic Equipment Decomissioning	34
Back Angle Adjustment	19	Appropriateness of Equipment	34
Back Height Adjustment	19	Manuals and Records	34
14.0 Seat Depth Adjustment	20	Inspection	34
Adjusting The Seat depth (Fig14.1)	20	Condition	34
15.0 Fitting A Tray	20	Accessories	35
Tray Fitting Instructions (Fig 15.1-15.2)	20	Function	35
16.0 Pad Adjustment	21	Fixings	35
Sacral Pad Adjustment (Fig.16.1)	21	Modifications	35
Hip Pad Adjustment (Fig.16.2)	21	Labelling	35
Thigh Pad Adjustment (Fig.16.3-16.5)	22	Adjustment	35
17.0 Headrest Adjustment	23	Hand Over	35
Adjusting The Wing Angle	23	Packaging	35
Adjusting the Headrest Angle	23	Handle Fitting	35
Adjusting the Headrest Height	23		

Definitions

Definitions of words used in this manual

Word	Definition
 DANGER!	Advice to the user of potential risk of serious injury or death if the advice is not followed
 WARNING!	Advice to the user of a risk of injury if the advice is not followed
 CAUTION!	Advice to user that potential damage to equipment may occur if the advice is not followed
NOTE:	General advice or best practice
	Reference To Additional Documentation

NOTE:

The seating system shown and described in this user guide may not correspond in every detail exactly to your own model. However, all instructions are still entirely relevant, irrespective of detail differences.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

NOTE:

Please keep a note of your local service agent's address and telephone number in the space provided.

In the event of a breakdown, contact them and try to give all relevant details so they can help you quickly.

Dealer signature and stamp.

1.0 Important Safety Advice

WARNING!

We at Sunrise Medical are committed to producing products of the highest standard. All of our products fulfil the essential safety and environmental requirements as defined in the European Directives. However, improper use of the products will potentially put the users at risk and therefore Sunrise Medical strongly suggest that the following information is strictly adhered to at all times.

WARNINGS!

- Ensure all adjustment mechanisms are secure and in place before operation. If it is likely that the hand-wheels will be repeatedly loosened, Sunrise Medical can supply allen key bolts as an alternative. We strongly recommend this if there is a danger from those in the vicinity of the user.
- After completing any alterations ENSURE all nuts, bolts, knobs, hand wheels and other fixings are securely tightened and in position, and that they are regularly checked as part of the maintenance of the chair. Never over adjust or over tighten moving parts.
- All postural support straps and harnesses should be in place and properly adjusted to the user, prior to usage of any kind (see pages 9, 10 and 11).
- When fitted to an indoor base frame the Pluton should always be positioned on a level, flat floor. The Pluton chair can be moved between one working area and another. For safety ALWAYS return the product to a neutral position and lower before moving (lower in height, level the seat, ensure the back is upright etc). After the chair has been moved and during use the chair should be placed in a static position with the brakes applied. Indoor base frames such as the multi height and tilt base are designed to be used inside, however it is acceptable to take these bases externally for a brief period if transferring the chair from one indoor area to another.
- Heavy items on the tray will affect stability. The fitting of anything other than the standard Sunrise Medical tray may substantially affect the stability of the seating system and should therefore be checked before issue.
- If at any time it is noted that areas of the users skin remain reddened after being out of the seating system for around 10 minutes, urgently contact the qualified professional who performed the hand over of the equipment. This may be a sign of excessive pressure being exerted by the seating. This might occur in the initial use of a new seat where further adjustment may be required, where the user has been badly placed, grown or where an underlying medical problem exists. Review may be necessary in such cases
- Keep all products away from excessive sources of heat, cigarettes and naked flames.
- If you suspect that the system may be faulty, cease use of the equipment straight away and immediately contact the organization who supplied the system. (contact information can be found on page 5).

- The equipment will be labelled with important information. NEVER REMOVE these information labels or allow them to be defaced, overlaid or altered.
- All modifications, adjustments, reconditioning, repairs, disposal, and servicing of the seating unit must ONLY be carried out by the agencies who supplied the equipment (see pages 30-31).

DANGER!

Multi Adjustable Head support

Wings on the head support are ideally set at 45 degrees. Do NOT bring the wings in tight to the head. Ensure that the head support is set in such a way that the child cannot loop their head around the wings and get stuck.

Flexi-Supports and Lateral Supports

Ensure the straps do not infringe on any feeding tubes the child may have.

WARNING!

Abductors

These parts are designed to abduct the users knees to help keep the hips in a good position. Do NOT use the abductors as a way of holding the user in the seat. Flip down the abductor when transferring the user in and out of the seat.

Foot Sandals and Straps

Should only be fixed when the user is wearing shoes or boots.

Tray

Do NOT use the tray as a way of securing children in a chair. Always ensure that the lap strap and any other straps supplied are used. Ensure that the tray clips are secure on both sides of the tray. If the tray is damaged, ensure that this will not be a hazard to the user. Never leave hot items on the tray while children are unattended.

DANGER!

CHOKING HAZARD – This mobility aid uses small parts which under certain circumstances may present a choking hazard to young children.

2.0 Warranty

GUARANTEE

THIS GUARANTEE DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.

Sunrise Medical* provides a guarantee, as set out in the warranty conditions, for wheelchairs to its customers covering the following

Warranty conditions

Should a part or parts of the wheelchair require repair or replacement as a result of a manufacturing and/or material fault within 24 months or for frame and cross-braces within 5 years after delivery to the customer, then the affected part or parts will be repaired or replaced free of charge. (Batteries are guaranteed for a period of 12 months subject to the correct battery care plan being followed)

To enforce the warranty, please contact Sunrise Medical Customer Service with the exact details of the nature of the difficulty. Should you be using the wheelchair outside the area covered by the Sunrise Medical customer service agent, repairs or replacement will be carried out by another agency as designated by the manufacturer. The wheelchair must be repaired by a Sunrise Medical designated Customer Service agent, (dealer).

For parts, which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the wheelchair in accordance with point 1).

For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, (following the fitting), in accordance with these warranty conditions.

Consumable parts such as pads, tyres, inner tubes and similar parts are excluded from the warranty, except in the case that premature wear of the part is the direct result of a manufacturing fault.

Claims from this warranty shall not arise, if a repair or replacement of a wheelchair or a part is required for the following reasons:

The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the User Instructions and/or the Service Instructions.

Accessories have been used which are not specified as original accessories.

The wheelchair or part having been damaged by neglect, accident or improper use.

Changes/modifications have been made to the wheelchair or parts, which deviate from the manufacturer's specifications.

Repairs have been carried out, before our Customer Service has been informed of the circumstances.

This guarantee is subject to the law of the country in which the product was purchased from Sunrise Medical"

* Means the Sunrise Medical facility from which the product was purchased.

3.0 Brakes

Operating Brakes, (Indoor Base Frames)

WARNING!

Always apply the brakes when the chair is stationary.

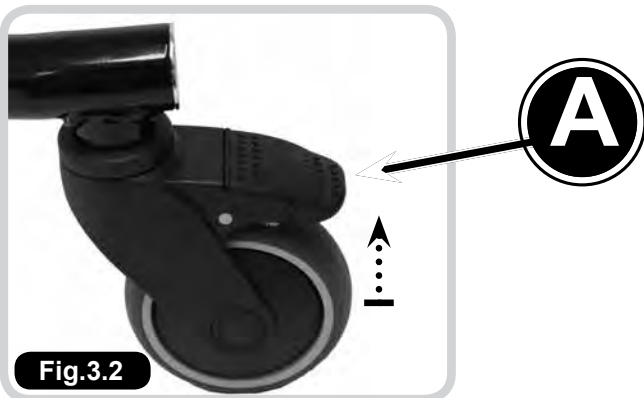
Applying Brakes

Depress lever (A) with your foot. To secure the chair in place you should suppress all the brakes fitted to the wheels on the base. (Fig.3.1).

Releasing Brakes

To release the brakes you should flick the levers up using the top of your foot. When all brakes are released the chair will be free to travel in any direction. (Fig.3.2).


4.0 Chest Harness-Adjustments

Chest Harness


DANGER!

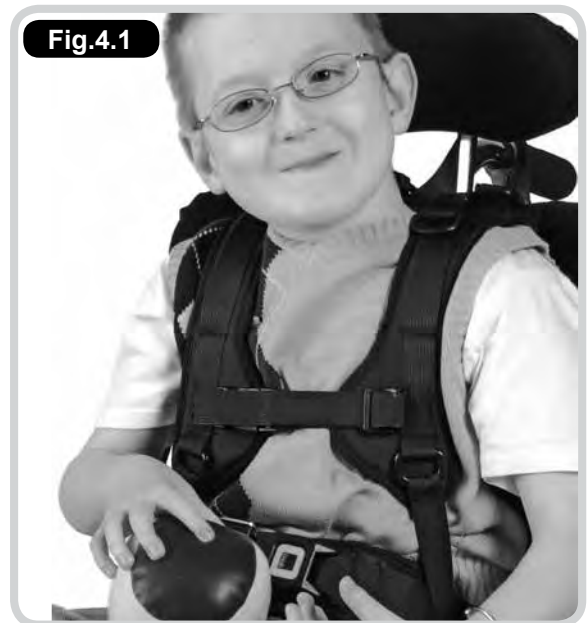- GREAT CARE should be taken when fitting a chest harness and the following points should always be followed:
- The straps on the harness should never be allowed to move close to or in any way cause an obstruction to the child's neck.
- The straps should never be fitted too tightly and should be clear from any obstruction.
- The sternum strap on the chest harness should always be adjusted prior to use to ensure that it is no higher than the middle of the user's chest.
- A lap strap should always be fitted if a harness is being used to ensure the child cannot slide down onto the sternum strap.
- Always ensure that the belts offer your child both comfort and support. If the child's clothing has been adjusted (i.e. a jumper removed) the straps should be re-adjusted accordingly. (Fig.4.1).


Fixing Onto Chair

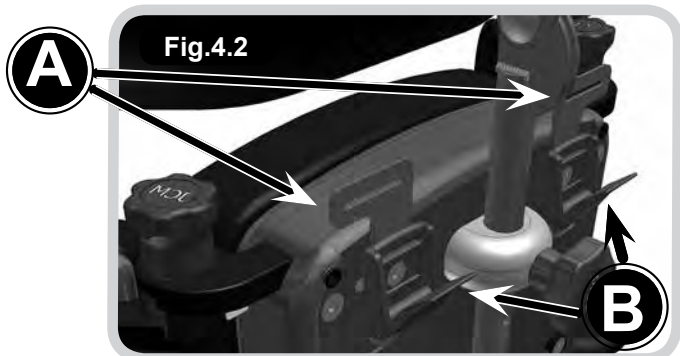- The Pluton comes with guide plates for the top straps which are located either side of the upper back (A). The webbing strap should be fed through these guides and then adjusted and secured in place using the cam buckle clasps (B) directly beneath the guide plates. (Fig.4.2).

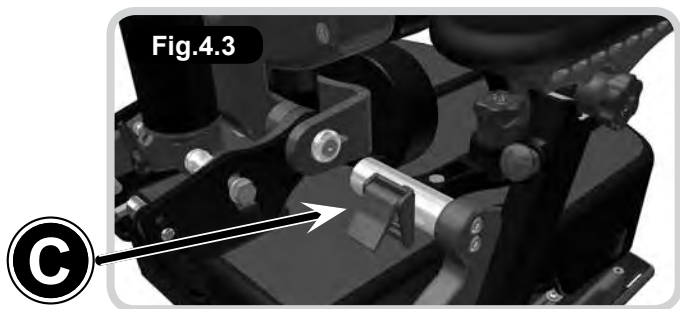- The straps at the bottom of the harness should be fed through, adjusted for fit and then locked using the Cam Buckle Clasp (C) located on the hip pad rail. (Fig.4.3).

Fastening of Belts

- Feed the fabric belt through the buckle on the strap.
- The belts should be pulled through enough to suitably support your child whilst still being comfortable.
- When this has been achieved the buckle should be snapped closed to secure the strap in place. (Fig.4.4).

⚠ WARNING!

The same precautions highlighted here should be adhered to when using any form of trunk supports or flexi supports with straps.

⚠ DANGER!

If there is any possible chance of the child getting the straps caught around their neck, REMOVE THE HARNESS IMMEDIATELY.

5.0 Lap Strap Safety

⚠ DANGER!

INCORRECT FITTING OF LAP STRAPS CAN PUT THE USER AT SERIOUS RISK.

Lap Strap Fitting

When using lap straps we recommend the following points should always be adhered to:

The angle that the hip belt is attached to the seating system has a direct effect on the angle of pull on the pelvis.

Considerations:

- A 60° angle of attachment to the seat rail positions the hip belt anterior and inferior to the ASIS, which reduces the tendency of the client with a posterior pelvic tilt to slip underneath the belt.
- A higher angle of attachment of the primary straps encourages a posterior pelvic tilt, which assists in positioning clients with an anterior pelvic tilt.
- The secondary straps of a 4-point hip belt are attached to the seat post between 45° and 90° to hold the primary padded strap in place and to prevent the belt from riding into the abdomen or twisting.
- The angle of attachment may have to be compromised in order to accommodate such situations as wounds in the pelvic area, unstable hip joints or gastric tubes.

Tightening the belt:

Keep belt tightened at adjustment straps during fitting and daily use to ensure correct pad placement. For padded hip belts the pads are to be touching when correctly tightened. For non-padded hip belts the adjustment strap at the buckle should be approximately 3" (76mm) long. Education of the care giver is essential for correct hip belt positioning.


Sizing:

Measure hip width to determine the belt size required. The suggested method for measuring hip width is from trochanter to trochanter. If this is not possible, measure from ASIS to ASIS and add a couple of inches. The correct belt size for the individual has padding that covers all of the bony prominences.

- Always check that the child's pelvis is symmetrical and positioned securely at the back of the seat.
- Always ensure that the lap straps and chest harness hold the child securely and are comfortable. Ensure they provide a snug fit, a simple rule of thumb is to allow two fingers to be inserted between the belt and the child's body.
- Never leave a child in a chair unattended without the lap strap being fastened.
- A lap strap should always be fitted if a harness is being used.
- When altering the angle between the seat and the back of the chair always re-adjust the chest harness and lap strap accordingly to ensure all safety recommendations are followed.

Harness and Lap Strap Positioning

This diagram below shows the recommended positioning of the chest harness and lap straps, (Fig.5.1):

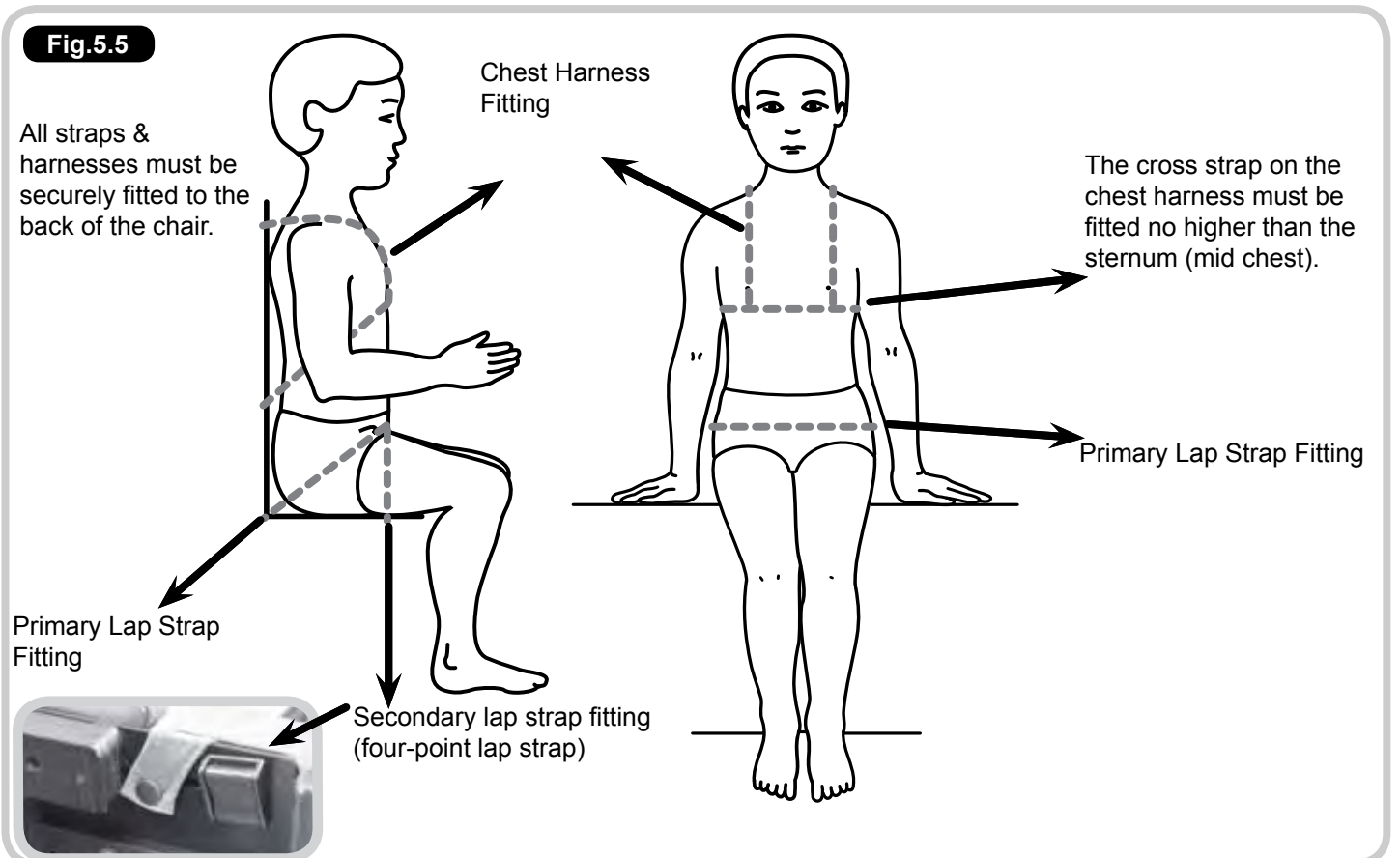6.0 Adjustment Points and Lever Use

Adjustment Points Underneath Seat, (Fig.6.1)


Fig.6.1

Seat depth adjustment bolts

Lap belt front fixing buckle

Footplate Angle Adjustment

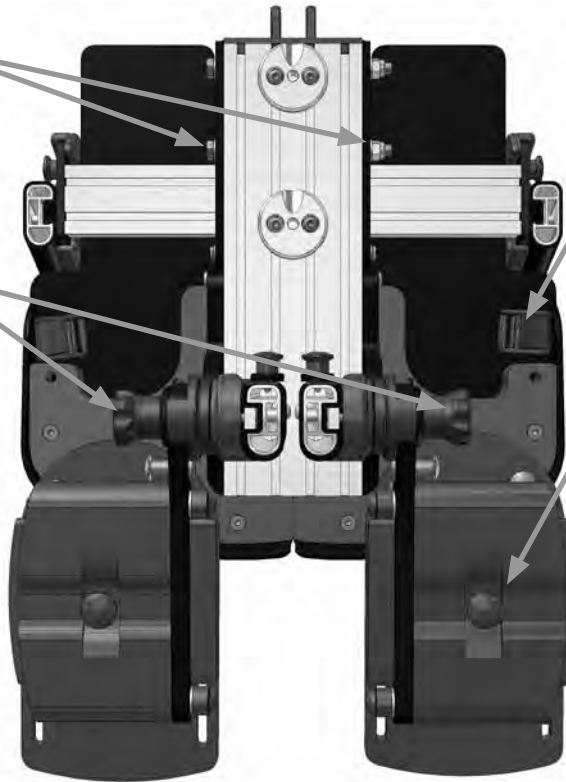Footplate width adjustment

Effective Operation of Lock Levers (Where Fitted)

Never remove these levers entirely as this will prevent you from being able to perform adjustments in the future.

- To use: Turn the lever handle clockwise to tighten or anti clockwise to loosen.
- To use the lever when an obstruction is met: Pull the handle of the lever outwards, towards you (A). This disables the function of the lever.
- Continue to hold the lever out and reposition the lever handle in a suitable position past the obstruction.
- Release the handle to re-engage the lever function (B) and continue to tighten or loosen in the normal way.
- Repeat this procedure in areas where adjustment is restricted due to an obstruction.

Fig.6.2

Fig.6.3

7.0 Lever Use-Using Index Plungers

Using The Index Plungers (locking Variety Only)

To place the plungers in an open position:

- Pull out the plungers to maximum tension (B).
- Turn the plungers a half turn anti-clockwise (C) This places them in an open position.
- To close - Turn the plungers a half turn clockwise and they should snap back into a locked position (A).

8.0 Adjusting Shoulder Protector Pads

Pad/ Shoulder Width

Turn locking screw (A) half a turn using the 4mm Allen Key provided, manually position the pad to fit the user by sliding the pad in or out.

Lock the pad in place by re-securing locking screw A.

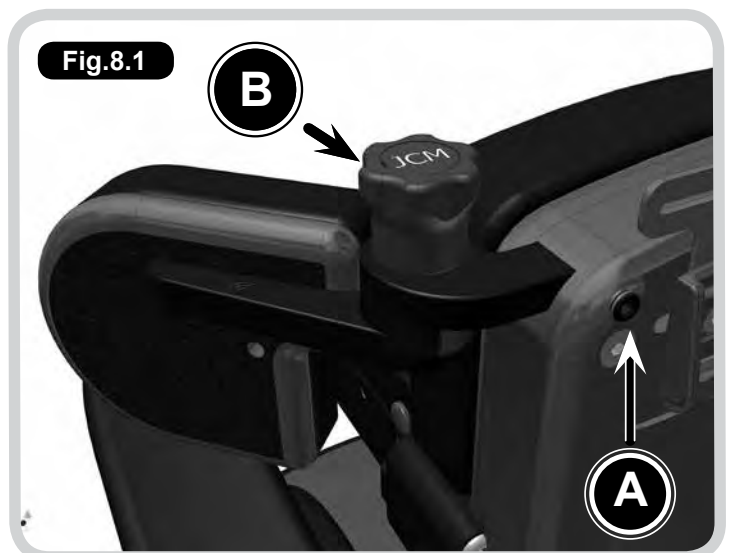
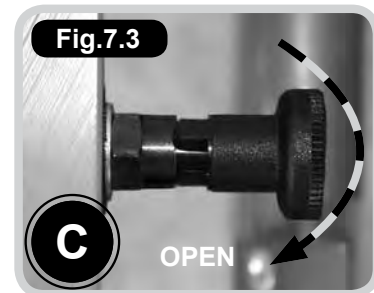
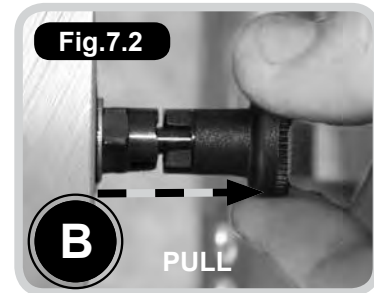
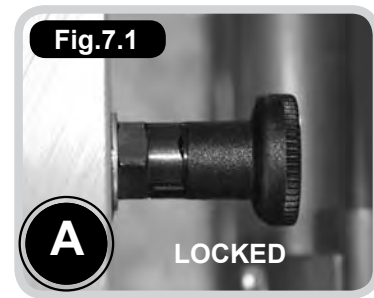
Pad/Protraction Angle

To set the angle of the pad you should loosen the securing handwheel (B) by rotating a couple of turns in an anti-clockwise motion. Once released the pads can be positioned by pressing down on the handwheel whilst simultaneously altering the angle of the pad using your other hand.

Once the desired position has been achieved you should release the handwheel and resecure it by turning in a clockwise direction until hand tight.

CAUTION!

Pad Angle – As an Alternative you can position the pad without pressing down the handwheel, though this has potential to shorten the life of the locking mechanism and is therefore not recommended.

9.0 Seat Unit

Mounting

WARNING!

Always ensure the base is stable and that the brakes are applied before mounting the seating unit.

CAUTION!

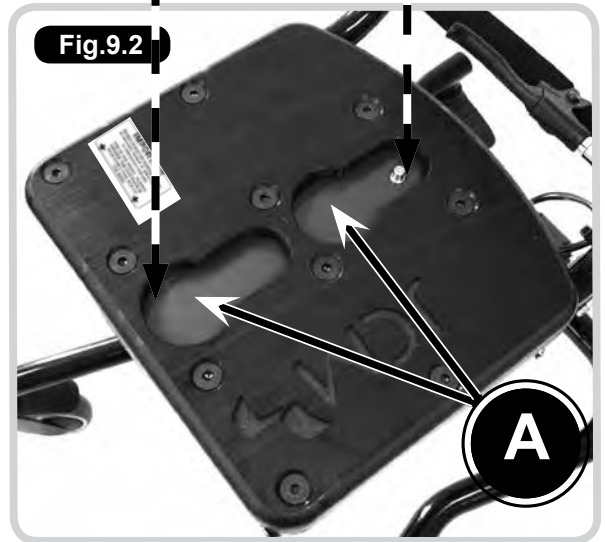
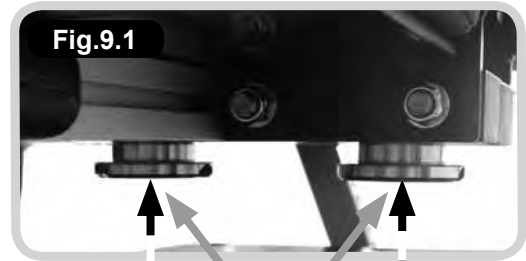
Please note that the handle fitted is dependent on the size of the chair. You therefore need to ensure the size label on the handle matches the size of the chair being used.

See, (Fig.9.1-Fig.9.2).

- There are 2 cut outs on the top of the interface plate (A) which are for locating lugs (B) on the seating unit to fit into.
- Position the seating unit above the interface plate, lining up the locating lugs (B) with the cut outs (A) on the interface plate.
- Carefully lower the seat onto the interface plate ensuring the locating lugs (B) slide into the cut outs on the interface plate (A).
- Push the seating unit inwards towards the back of the chair until the locating lugs 'click' into place.
- Ensure the seat is securely located on the base before using.


View from the side showing the seating unit lugs lowering into the interface plate cut outs, (Fig.9.3).

View of seat engaged with the base (Fig.9.4).

Removal

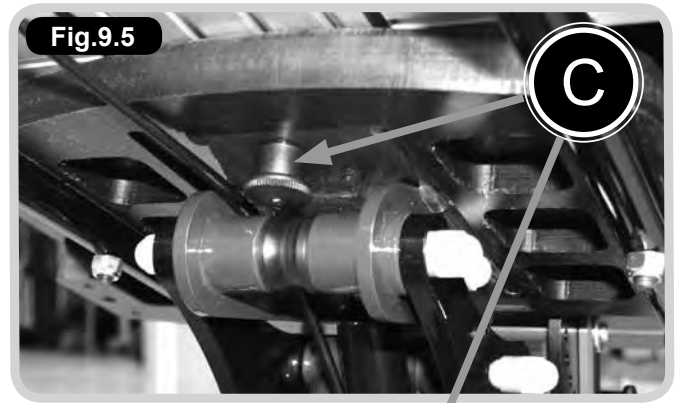Release index plunger (C) which is underneath the interface plate and slide seat forward to remove it from the base, (Fig.9.5-9.6).

Manual Seat Height Adjustment

Multi height & tilt base

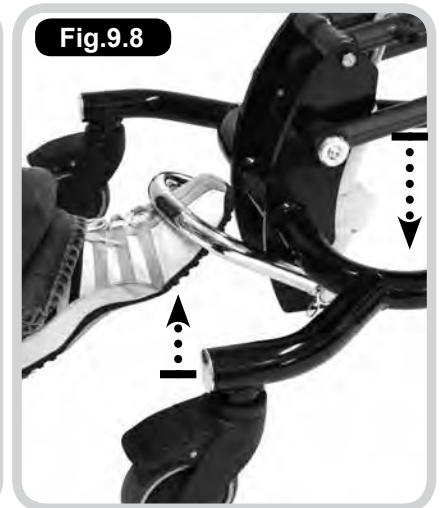
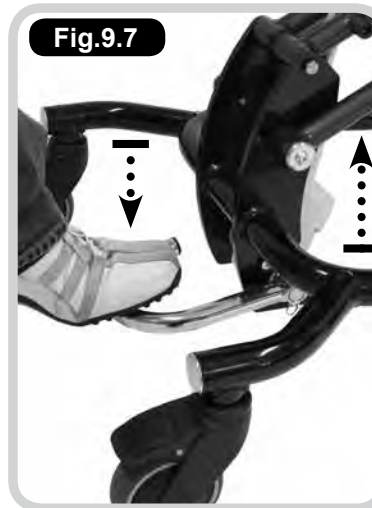
The multi height and tilt base caters for all sizes of Pluton. It is controlled by a hydraulic pump mechanism.

To Raise The Chair, (Fig.9.7).

Pump the foot pedal several times until the desired height is reached. (The chair will raise with each pump.)

To Lower The Chair, (Fig.9.8).

Place your foot under the pedal and gently lift the pedal up. (The chair will automatically lower).

Powered Base Height Adjustment

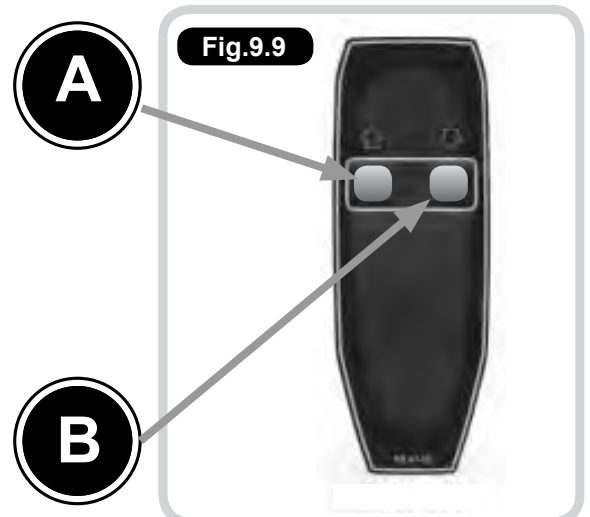
Upgrade option, (Fig.9.9).

CAUTION!

Always ensure that adjustment mechanism is free from obstruction before operating this chair.

Button A = UP

Button B = DOWN

10.0 Tilt In Space

Multi Height & Tilt Base (Fig.10.1-10.3)

WARNING!

Always ensure the base is stable and that the brakes are applied before performing the following adjustments.

- Release the locking mechanism on the handle by squeezing the small latch (A) up on the handle.
- To Recline:
Use one hand to squeeze handle (B) and push down the chair handle, at the same time use the other hand positioned on the chair arm or side to help guide the chair into the recline position.
- To put into Prone:
Squeeze handle (B) and pull the chair handle upwards and forwards. At the same time use the other hand positioned on the chair arm or side to help guide the chair into the prone position.
- Once in position release handle (B) to lock the tilt mechanism.
- Ensure that the latch (A) on the handle moves back into a closed position, this ensures the tilt position is locked in place.

WARNING!

The operator should control the motion of the chair while the squeeze handle is engaged. The amount of support required will vary with the size and weight of the user. We recommend practicing this operation on first use before placing the client in the chair.

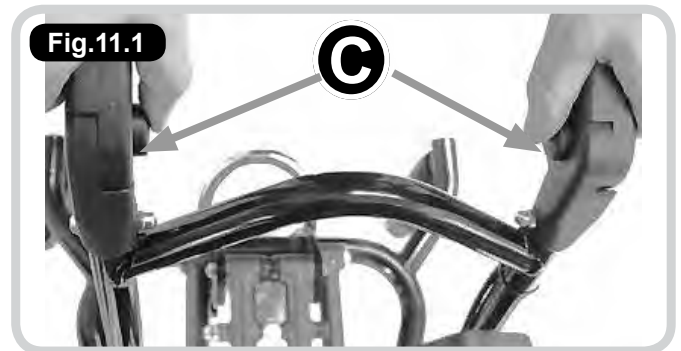
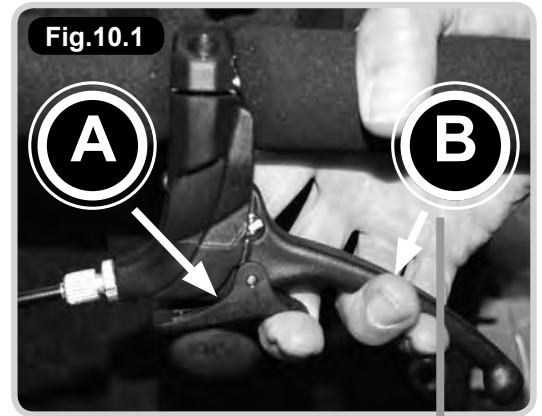
11.0 Folding Push Handle

Multi Height and Tilt Base (Fig.11.1-11.2)

Only use the push handle to move the chair. The push handle should be folded down when not in use.

Depress both buttons (C) on the inside of the handle. Manually alter the position of the handle by pushing the top section down.

To unfold pull the handle up until you hear a click ensuring it is in place.

12.0 5-Star Base Adjustment

WARNING!

Always ensure the base is stable and that the brakes are applied before performing the following adjustments.

Refer to Fig.12.1.

To Raise The Chair

Located at the back of the chair beneath the base locate the handle (A) and continuously push down to raise the height.

The seat height will raise with additional assistance required, dependent upon the weight of the user. Release the handle when the desired height is achieved.

To Lower The Chair

Use the handle (A) again but this time apply pressure to the top of the seat with one hand whilst pushing down handle (A).

CAUTION!

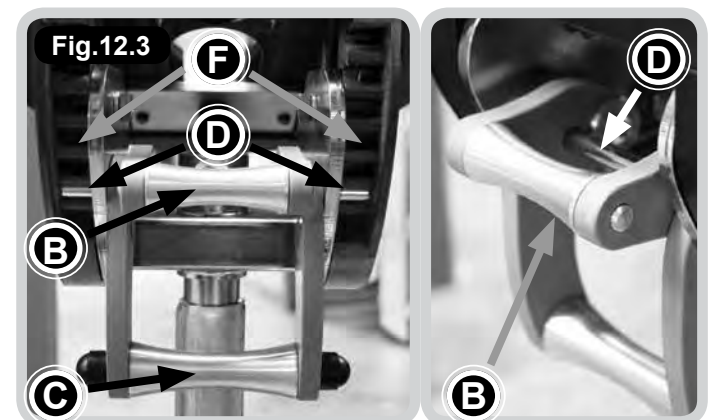
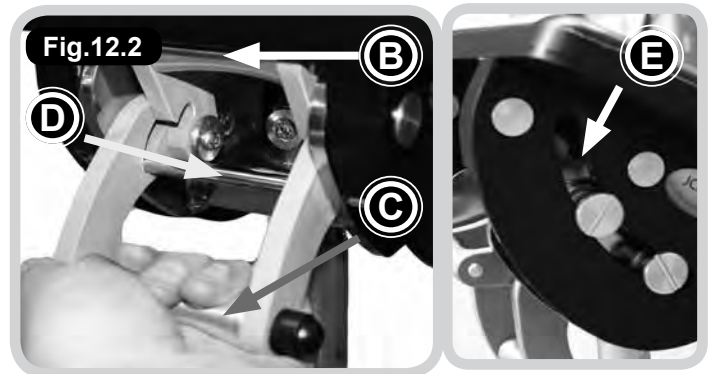
If you find performing these actions difficult to do simultaneously on your own please get another person to help you.

To Perform the Tilt in Space on the 5 Star Base

Refer to Fig.12.2-12.3

- Lift safety latch (B) up to prepare the base for the tilt adjustment.
- Lift operation latch (C) up until it stops, disengaging bar (D).
- Tilt the base interface to the desired angle. The adjustment range can be seen via the groove at the side of the seat base. When the desired angle has been achieved. Close operation latch (C) by pushing down until it bar (D) locates securely into a set position groove (F).
- Push down safety latch (B) to prevent accidental operation of the tilt in space.
- Always ensure that the groove on the safety latch (B) is securely located onto the bar (D) to secure the seat into place.

13.0 Back Rest Adjustment

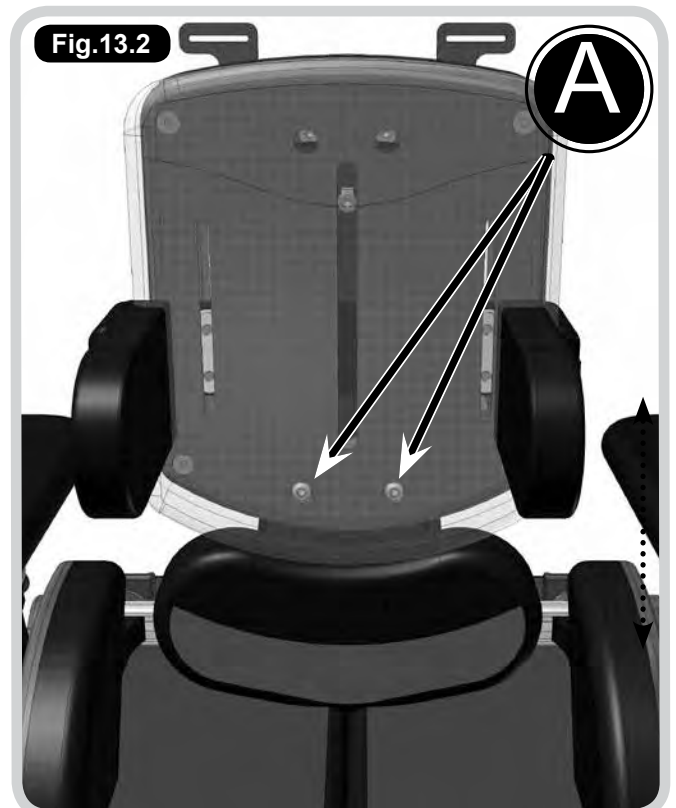
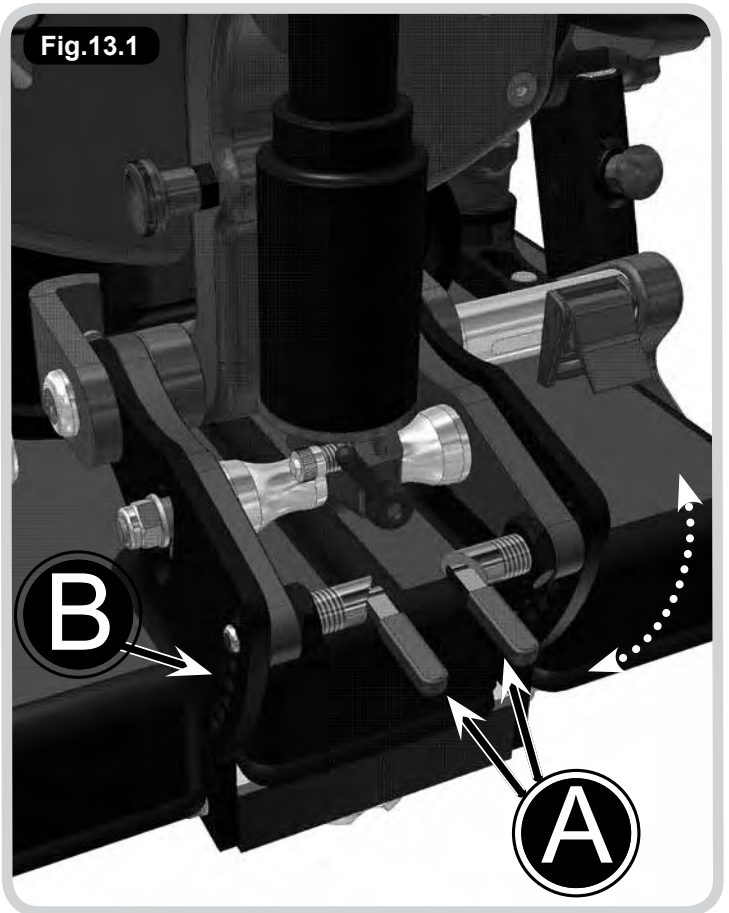
Refer to Fig.13.1-13.2

Back Angle Adjustment

- At the lower back of the chair squeeze together levers (A), this releases them from their holding position in holes (B).
- Whilst holding the levers together, hold the chair by the top of the back and guide the angle of the back into the required position.
- Relocate the levers into the relevant positioning holes (B) to secure the back in place.
- Always ensure that the back is secure and that the levers are located fully in a hole before using the chair.

Back Height Adjustment

- Remove the back pad cushion which is held in place by press studs.
- Loosen the two allen key bolts (A) on the plastic back panel.
- Slide the plastic panel up or down to the desired height.
- Tighten the two allen key bolts securely.
- Replace the back cushion pad.

14.0 Seat Depth Adjustment

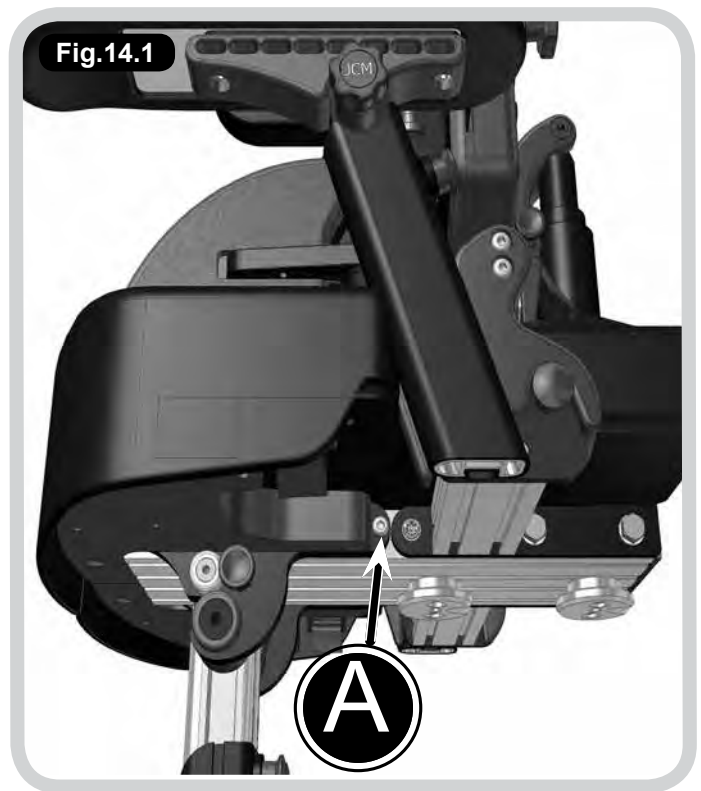
Adjusting The Seat depth (Fig14.1)

WARNING!

- When mounting the seating system to a base for the first time, or when an adjustment has been made to the size of the system, a stability assessment should be carried out prior to use of the equipment.
- Once you have adjusted the seat length, always ensure these points are firmly fixed and are checked as part of maintenance of the chair.

NOTE: The Pluton Seat is Split into 2 halves (Left & Right) Follow the instructions below to adjust the seat depth for the user. If the user has a leg length discrepancy then you can adjust and lock off either sides of the seat at any point within their range (150mm)

- Loosen (do not remove) the allen key bolts (A) on both sides of the seat.
- Slide the seat in or out to change the length.
- Fix the position by tightening the allen key bolts securely.

15.0 Fitting A Tray

DANGER!

- Always ensure that the arms are fixed in position and the tray is secure before leaving the user unattended.
- Never leave anything hot on the tray while the client is unattended.

Tray Fitting Instructions (Fig 15.1-15.2)

NOTE: Arms (optional accessory) are required in order to fit a tray.

- Pick up the tray from the front, holding from the top with the fingers of each hand engaged with the pull latches A on either side of the tray.
- Place the tray onto the arm tops B, sliding the tray in toward the occupant until the desired position is reached.
- Release the latches (sprung loaded) to engage them, ensuring the latch has fully locked into the closest available engagement position C on the underside of the Arm Top.
- Ensure both left and right latches are engaged and the tray is secure before commencing use of the chair.

Fig.15.1


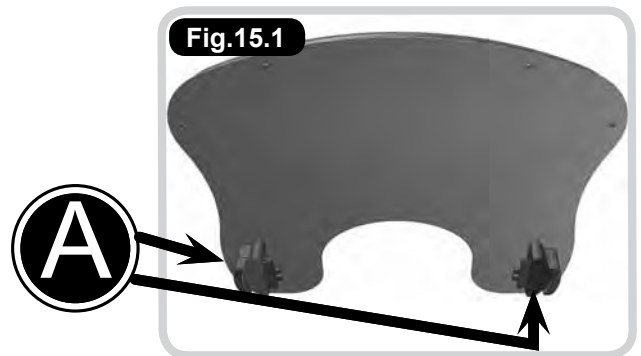
Fig.15.2

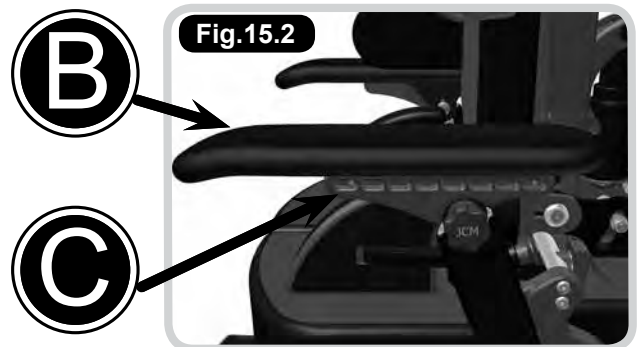16.0 Pad Adjustment

Sacral Pad Adjustment (Fig.16.1)

Pad Height

- Pull the cushioned pad away from the framework
- Loosen the allen key bolts (A) on both sides of the sacral pad (2 altogether).
- Move the pad up or down within the adjustment slot.
- Once in place tighten the bolts to secure.

Pad Angle & Depth

- Loosen Locking Bolt B by one full turn using the 4mm Allen Key Provided (Ensure this is complete for both left and right fixings)
- The Sacral Pad can now be freely moved for angle and fore & aft position.
- Set the desired position and then re-lock the fixing screws to secure the pad in place.
- Note: The Sacral Pad can also facilitate a degree of rotational adjustment – To achieve this you should adjust the fore & aft position of left & right sides independently.

Hip Pad Adjustment (Fig.16.2)

Hip Pad Width

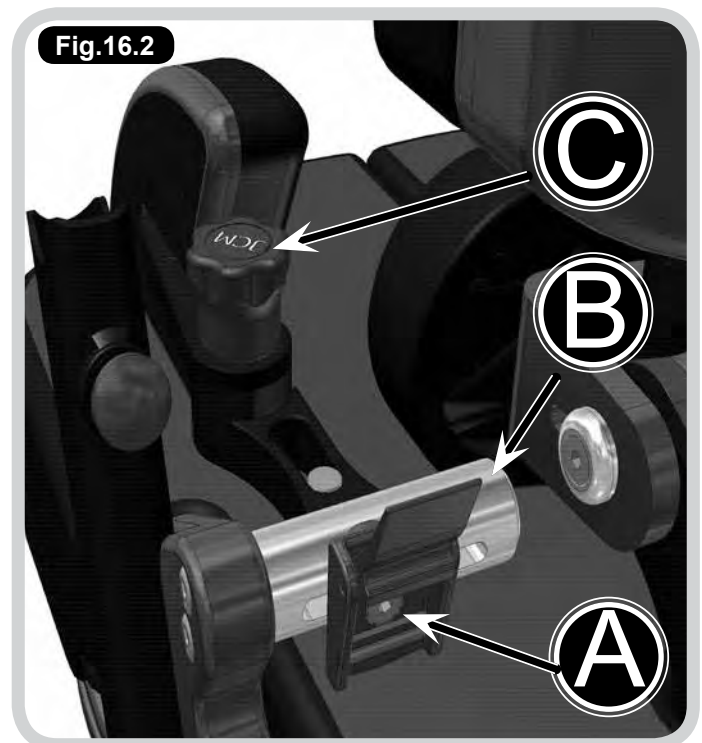
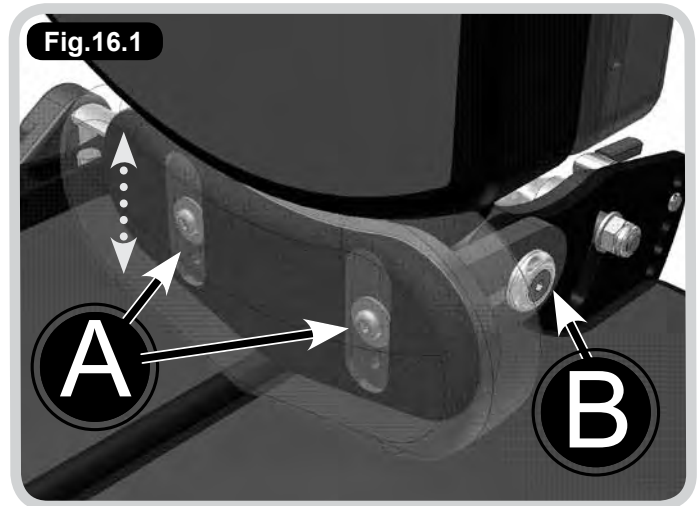
- Loosen Locking Bolt A by one full turn using the 4mm Allen Key Provided
- The Hip Pad can now be freely moved along the width adjustment rail B.
- Set the desired position and then re-lock the fixing screw to secure the pad in place.

Hip Pad Angle (Where Fitted)

- To set the angle of the pad you should loosen the securing handwheel C by rotating a couple of turns in an anti-clockwise motion.
- Once released the pads can be positioned by pressing down on the handwheel whilst simultaneously altering the angle of the pad using your other hand.
- Once the desired position has been achieved you should release the handwheel and re secure it by turning in a clockwise direction until hand tight.

CAUTION!

Pad Angle – As an Alternative you can position the pad without pressing down the handwheel, though this has potential to shorten the life of the locking mechanism and is therefore not recommended.

Thigh Pad Adjustment (Fig.16.3-16.5)

These pads enable you to sit the client in an abducted position which can help to lower muscle tone, create a wider sitting base and maintain the hips in a good position.

Inner Thigh Pad Fitting & Adjustment

- To fit the inner thigh pads, Screws A complete with Washers B should be inserted through the carrier plate and into the dedicated holes on the underside of the Seating Unit.
- Tighten each screw using the 4mm allen Key provided.
- Once held on (but not fully locked off) you can continue to set the other adjustment.

Abducting the legs

- With Screws A still loose, rotate the inner thigh pad to the desired position.
- Once attained, lock off screws using the 4mm Allen Key provided.

Flip Down the Inner Thigh Pad (To facilitate Transfers)

- Press and Hold Button C, whilst flipping down the inner thigh pad.
- To reengage the inner thigh pad, simply rotate back up and into place until it latches securely.

Outer Thigh Pad Fitting & Adjustment

- To fit the outer thigh pads, Screws A complete with Washers B should be inserted through metal adductor bracket into the dedicated holes on the plastic carrier plate.
- Tighten each screw using the 4mm Allen Key provided.
- Once held on (but not fully locked off) you can continue to set the other adjustment.

Adducting the Legs / Setting Channel Width

- With Screws A still loose, slide the outer thigh pad along it fixing slot to the desired position.
- Once attained lock off screws using the 4mm Allen Key provided.

Fig.16.3

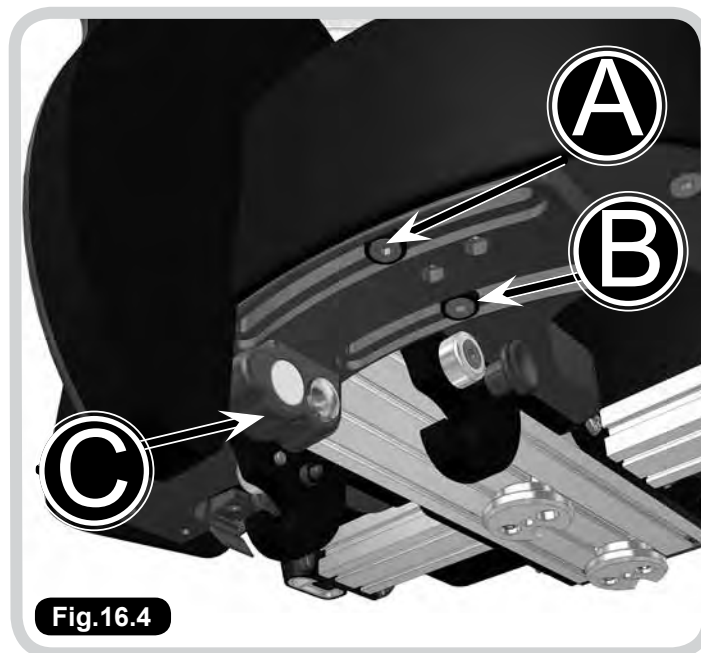
Fig.16.4

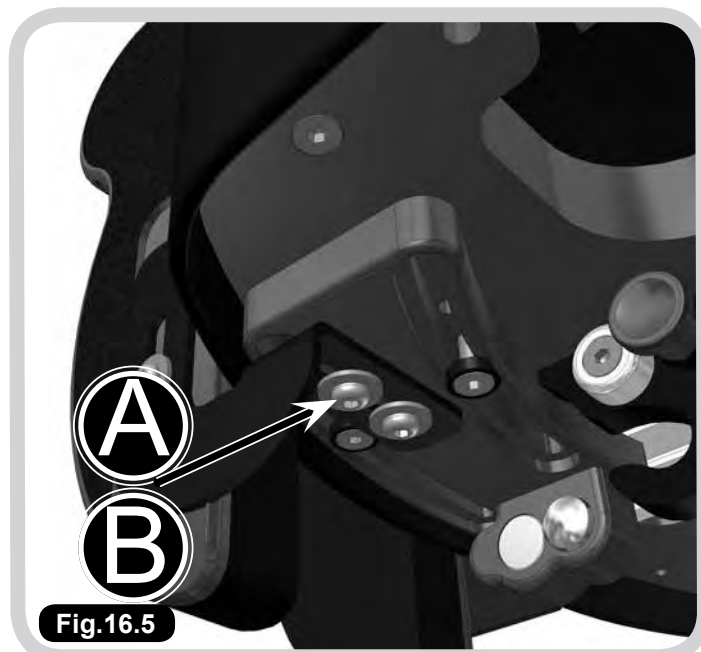
Fig.16.5

17.0 Headrest Adjustment

Adjusting The Wing Angle

This adjustment point allows you to move the wing position to offer more side support to the head. This adjustment point allows you to alter the angle of the headrest to support the head and neck.

- Unzip the headrest cover at the back and loosen the lock lever (A).
- Adjust the sides of the headrest into the desired position.
- After desired position has been reached tighten the bolts to secure in place.

Perform this for either side wing of the headrest.

Adjusting the Headrest Angle

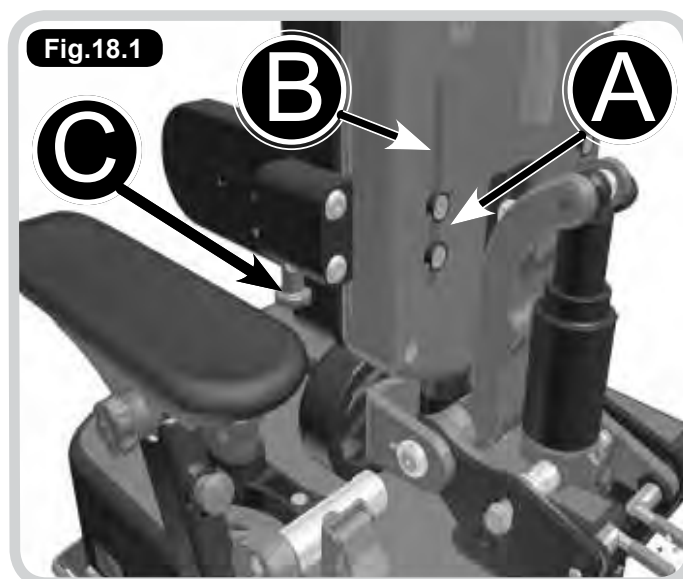
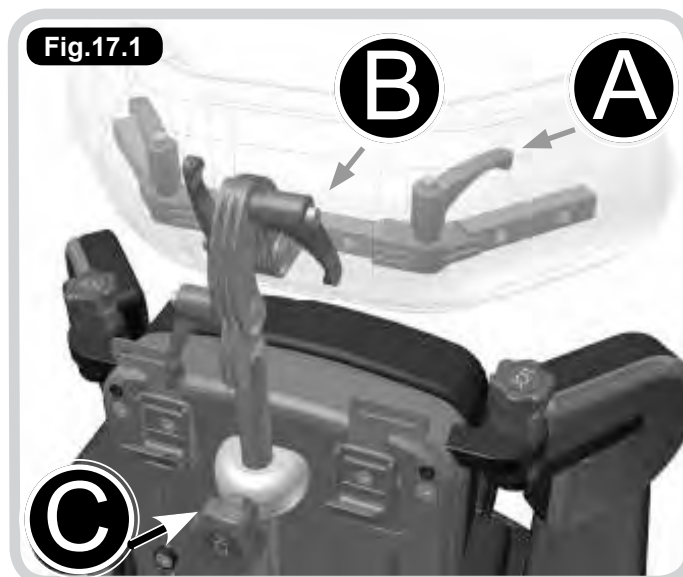
- Levers (B) should be operated together to achieve the desired head positioning of the user. Loosen the levers by turning anti-clockwise.
- Once loose, the head support angle can be manually adjusted to suit the user.
- Tighten the levers to secure in place.

Adjusting the Headrest Height

- Loosen the hand wheel (C)
- Slide the headrest up or down inside the bracket on the back of the chair.
- Re tighten the hand wheel to secure in place.

WARNING!

Always check that the headrest is secure and the wings are not tight around the head. The headrest wings are ideally set at 45 degrees to the back.

18.0 Lateral Support Adjustment

Height & Width of Supports

- Loosen both bolts (A) using the allen key provided.
- Slide the pads up or down along the slot in the seat back (B) to adjust the height.
- Slide the pads horizontally to adjust the width.
- Re tighten the allen key bolts to secure.

Flip Away Action (Where Fitted)

- Pull plunger (C) downwards.
- Flip the supports away to aid transfer or set up.
- When ready push the supports back in, until they lock in place.

WARNING!

Check that the supports are not too high under the arms of the user or tight around chest area by running your fingers around the inside of the supports. Also check that the supports do not infringe on any feeding tubes.

19.0 Arm Rest Adjustment

Adjusting Arm Height

- (A) highlights the plunger which is used to adjust the height of the arms and therefore in turn the height of the tray.
- Pull plunger (A) outwards away from the arm, and while holding the lever open slide the arm up or down, selecting one of the holes which come into view as you raise the arm up from its lowest position.
- When a suitable position has been acquired, lock the arm into place by releasing the lever, ensuring it fits securely back in place.

Adjusting Arm Angle

- To set the angle of the arm top you should loosen the securing handwheel (B) by rotating a couple of turns in an anti-clockwise motion.
- Once released the arm top can be positioned by pressing down on the handwheel whilst simultaneously altering the angle using your other hand.
- Once the desired position has been achieved you should release the handwheel and re secure it by turning in a clockwise direction until hand tight.

CAUTION!

The plunger (A) should pull out freely, if resistance is felt ensure no weight is being put through the arm rest.

Removing The Armrest

(Facilitates Side Transfer)

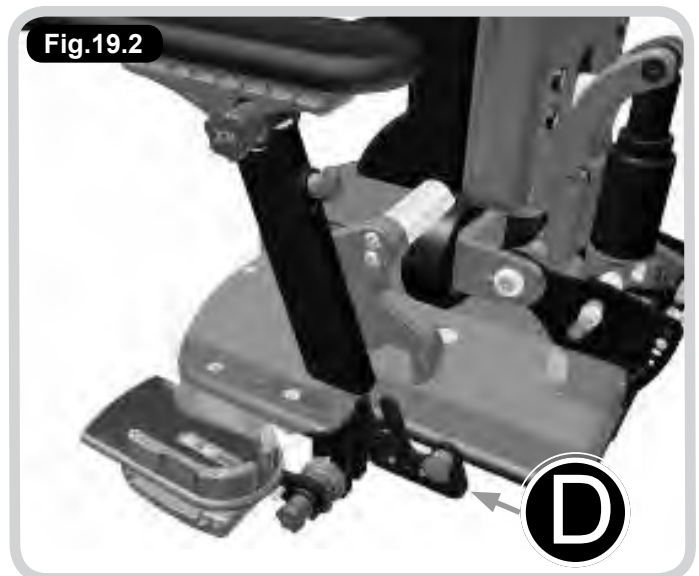
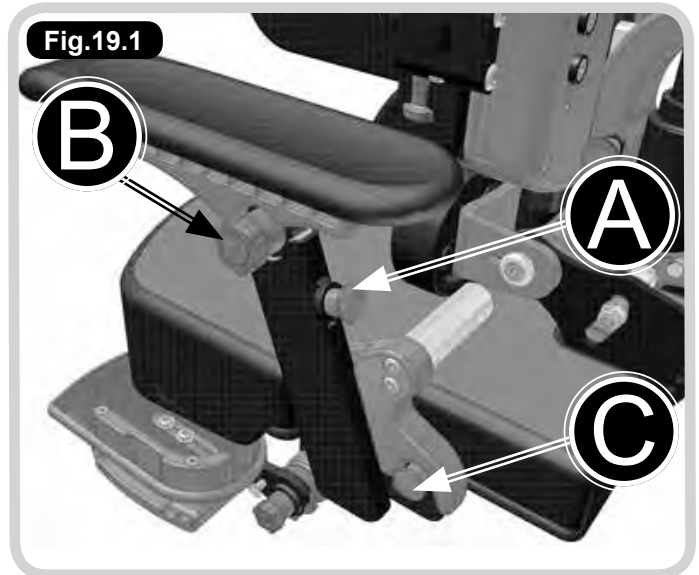
- To remove the arm rests entirely (complete with the attached hip pad), pull plunger (C) outwards and hold in the outward position with one hand. Use your other hand to lift the complete arm rest assembly upwards away from the seat.
- To replace, reverse the operation, lowering the arm rest assembly back onto the fixation plate (D), holding open the plunger to allow it to engage fully before releasing the plunger to lock the arm back into position.

WARNING!

Always ensure that the arms are securely fixed before using the chair.

CAUTION!

Arm Top Angle – As an Alternative you can position the Arm Top without pressing down the handwheel, though this has potential to shorten the life of the locking mechanism and is therefore not recommended)

20.0 Legrest Adjustment/Removal

Split Option:

NOTE: When fitted with Split Leg Stems/Footplates, it is possible to set the angle for both left and right stems independently. To do this follow the instructions below for each leg stem.

Angle Adjustment

- Pull and Hold Out plunger A with one hand before adjusting the legrest angle with your other hand to the desired angle.
- When you have achieved the desired position, release the plunger to secure in the next available locking position.
- Ensure the plungers are securely located in position.

Removing the Leg Stem

Facilitates Standing Transfer.

NOTE: Removal of the Footplates can also help when packaging for transportation or when the seat is to be utilised on a chassis with its own integrated footplates.

- To Remove the Leg Stems Entirely (complete with the attached footplate), remove safety boss B using the 4mm Allen Key Provided.
- Once removed, Pull and Hold Out Plunger A with one hand, using your other hand to rotate and pull away the Leg Stem entirely from the main seating unit.
- To replace, reverse the operation, locating the leg stem onto the fixation Hook C, holding open the plunger to allow it to engage fully before releasing the plunger to lock the leg stem into position.
- Finally refit the safety boss using the 4mm Allen Key provided.

Fig.20.1


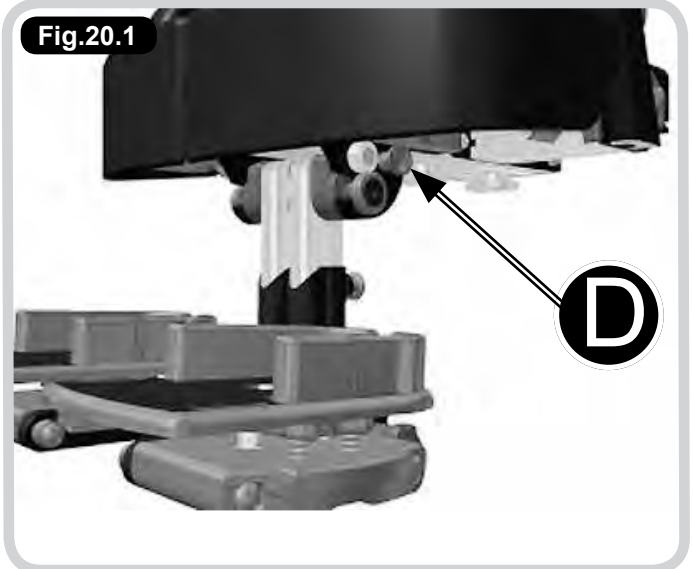
Fig.20.2

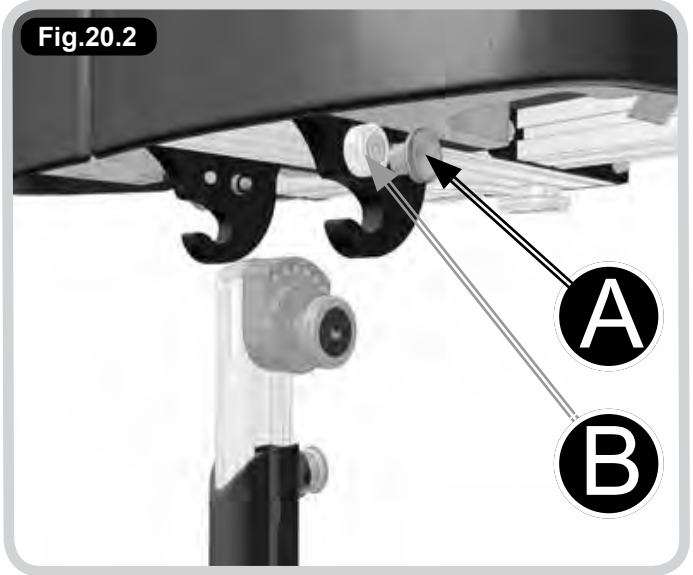21.0 Foot Position Adjustment

Footplate Angle Adjustment

- To set the angle of the footplate you should loosen the securing handwheel A by rotating a couple of turns in an anti-clockwise motion.
- Once released the Footplate can be positioned by pressing down on the handwheel whilst simultaneously altering the angle using your other hand.

Once the desired position has been achieved you should release the handwheel and re secure it by turning in a clockwise direction until hand tight.

Flipping Up the Footplate

Facilitates Standing Transfer.

- Press in Button B with one hand, using your other hand to flip up the footplate.
- To flip back down, simply rotate the footplate down until it latches back into place.

Footplate Height

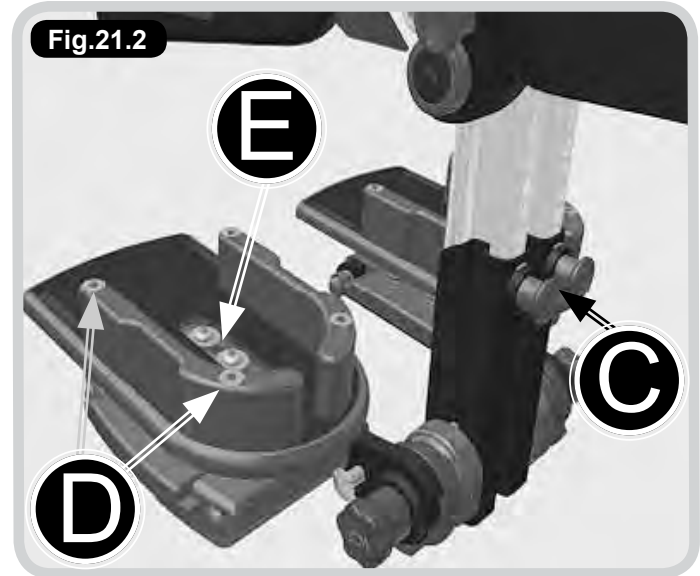
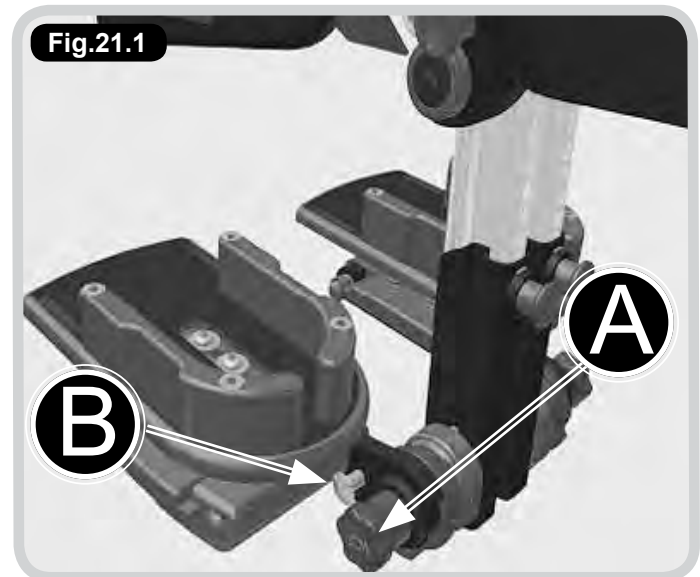
- Pull and hold open Plunger C, whilst maintaining hold of the footplate with your other hand.
- Once disengaged, the footplate stem should be free to slide up and down the legrest stem.
- Slide to the desired position before releasing the Plunger to engage it within the closest available location hole.
- Ensure the plunger is fully engaged before commencing use of the chair.

Foot Holder Positioning

NOTE: The Foot Holder/Plate can be adjusted for Foot Size, Fore & Aft, Side to Side (width between feet) and Foot Angle (in conjunction with Ab/Adduction) Each of these adjustments is covered below.

Foot Size

- Turn locking screws D anti-clockwise half a turn using the 4mm Allen Key provided
- Slide the Heel guide in or out to narrow/widen the space available for the foot.
- Repeat for the opposing Heel Guide if require before re-securing the locking screw D to hold them in place.


Fore and Aft, Rotation

- Turn Locking Screws E anti-clockwise half a turn using the 4mm Allen Key provided.
- Slide the entire foot holder forwards or backs whilst simultaneously setting the rotation of the plate.
- Once the desired position has been acquired re-secure the locking screws to hold back in place.

Side to Side/Width Adjustment

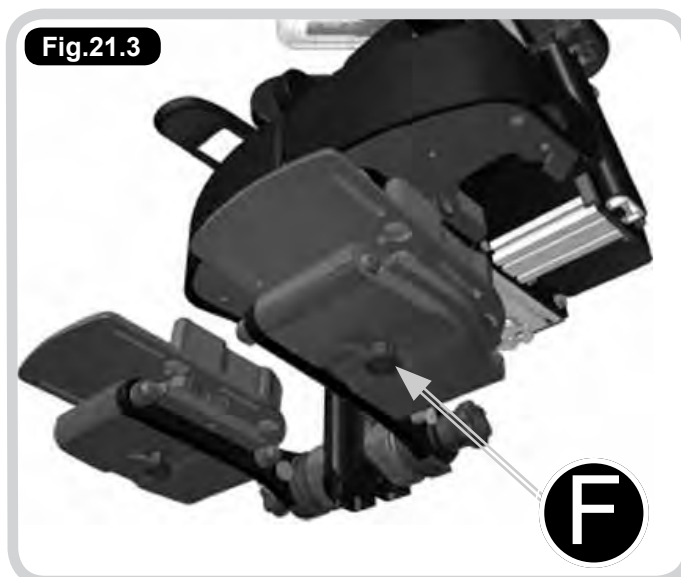- Pull and hold open Plunger F, whilst maintaining hold of the footplate with your other hand.
- Once disengaged, the footplate holder should be free to slide sideways, bringing the feet together/apart as required.
- When you have achieved the desired position, you should release the Plunger to engage it within the closest available location hole.
- Ensure the plunger is fully engaged before commencing use of the chair.

Foot Straps

Foot straps or ankle huggers can be attached through the slots in the sandal to secure feet. Feed the strap through the buckle and close the buckle shut to secure.

WARNING!

Do not use foot straps unless the client is wearing shoes.

CAUTION!

- Plunger C should pull out freely, if resistance is felt ensure no weight is being put through the footrest.
- Footplate Angle – As an Alternative you can position the Footplate without pressing down the handwheel, though this has potential to shorten the life of the locking mechanism and is therefore not recommended.

22.0 Dynamic Back Set Up

DANGER!

- Never modify your shock system. Modification, improper service or use of non-Sunrise Medical spare parts voids the warranty and could cause a malfunction resulting in serious injury.
- Follow the maintenance schedule, if the shock system shows any malfunctions such as losing oil, making unusual sounds or any part is bent or broken, cease use of the seating system immediately and contact Sunrise Medical for assistance.
- Never try to open or disassemble the shock system. Opening the shock system could cause serious injury.

Setting the Strength of Resistance & Power of Return

For maximum performance and comfort, it is necessary to adjust the strength of resistance and power of return of the dynamic mechanism, (A). The resistance is adjusted by varying the air pressure and should be determined by the user's weight, seating position, seat design and personal preference.

The air pressure is adjusted with a standard schrader valve shock system pump (B) on the main air valve of the dynamic mechanism.

Strong Resistance:

Increasing the Air Pressure

The shock system is more difficult to compress during use. The seat back feels firmer when pushed against.

- Attach the pump to the air valve on the back of the dynamic mechanism (B).
- Pump air into the valve to increase pressure.
- Watch the pressure valve dial on the pump to determine the amount of pressure added.

Weak Resistance:

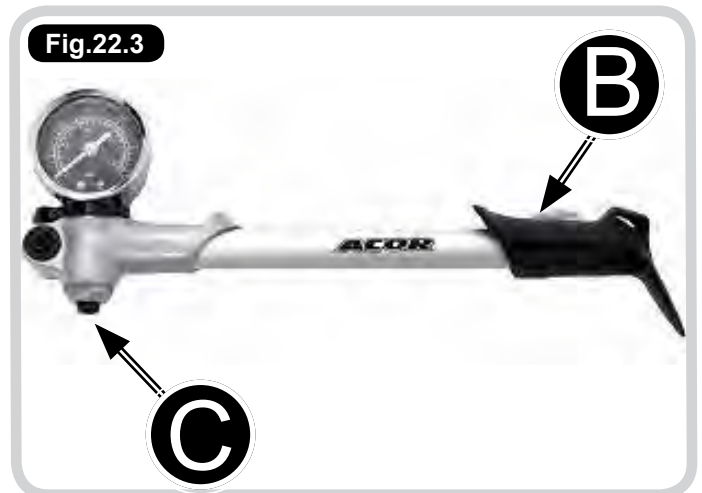
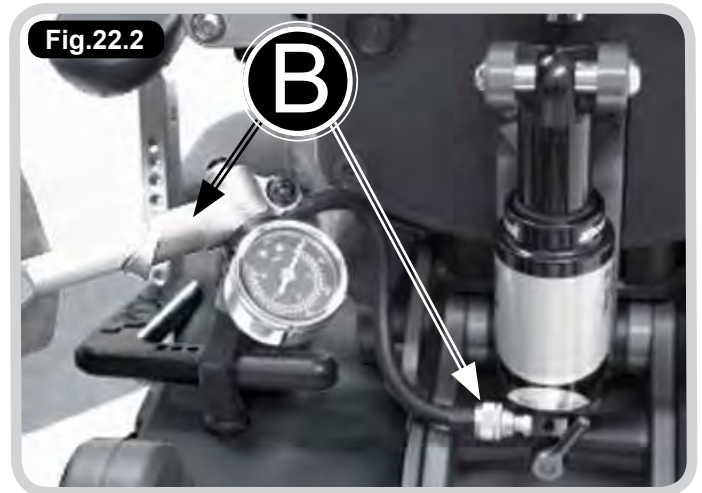
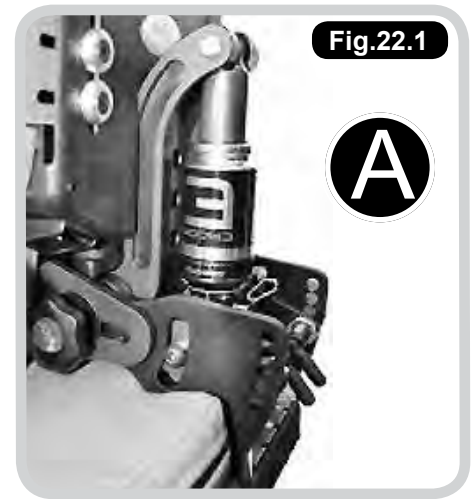
Decreasing the Air Pressure

Makes the shock system compress more easily. The seat back feels softer when pushed against.

- Press button (C) on the pump and keep depressed until the air pressure resistance achieves required level.

DANGER!

- The pressure range is 50-250psi.
- Never exceed an air pressure of 250psi in the uncompressed shock system.
- Exceeding the maximum air pressure of 250lbs/inch could result in serious injury.

Setting The Power of Return

The power of return controls the rate at which the shock system decompresses or rebounds. The amount of return strength needed depends on the user's weight, the spring rate which is set by the air pressure, the amount of use, the sitting style and personal preferences.

The best way to adjust the return strength is to decrease it completely by turning the red adjuster wheel (D) clockwise (labelled slow, "E") and test by placing the user in the seat. Increase the damping by slowly turning the wheel counter clockwise (labelled fast, "E") until the user feels comfortable and safe without the suspension kicking the user too far forward.

Disengaging the Mechanism

Occasionally at meal times etc it is necessary to disengage the dynamic mechanism. To do this:

- Lock the system by switching the lever from the "open" to "locked" position on the lever on the shock system, (F).
- The mechanism should not be left in the 'locked' position as part of normal use. You should ensure the mechanism is returned to the 'open' position once the activity demanding the lock out has been completed.

Maintenance Schedule

The rear shock systems need a minimum of maintenance only use water and soap to clean the system.

CAUTION!

Never use a high pressure washer or any water under pressure to clean the shock system as water may enter the system.

Before every use:

- Check air pressure (it is normal for the shock system to lose air pressure slightly over time) - Check to ensure it is appropriately set for your client.
- Adjust power of return set up to ensure it is set appropriately for your client.

WARNING!

- The extensive service of the shock system internals must only be done by an authorized technician.
- Please note that your Sunrise Medical rear shock is filled with oil and nitrogen. This makes it impossible to open the rear shock without having the knowledge and special tools required to carry out this task. Please do not try to open the rear shock for service issues, this implies a very high risk of getting seriously injured. Besides this you will not be able to reassemble the shock any more. Opening the shock will void its warranty.
- If there is a problem with your rear shock, cease use of your chair immediately and contact Sunrise Medical Seating Solutions.

Fig.22.4

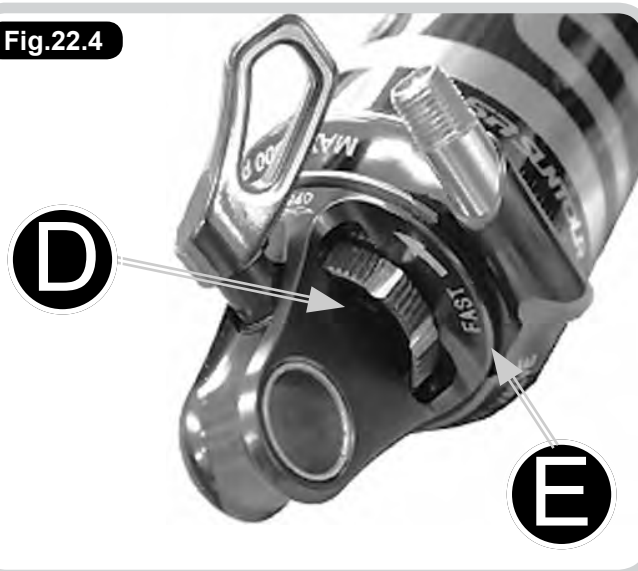
Fig.22.5

23.0 Cleaning and Care

We recommend the following points for successful cleaning of your equipment:

NOTE: It is possible that some surface wrinkling may occur following these processes.

Fabric Upholstery Care

Refer to Sunrise Medicals separate fabric cleaning guide which can be found on their website www.SunriseMedicalseating.co.uk or in hardcopy format from the Sunrise Medical office. Do not use bleach, acidic, solvent, abrasive, biological or phenolic based cleaners.

Metal Framework

- If all upholstery is removed the metal framework can be cleaned with hot soapy water and detergent or it can be steam cleaned. (Labelling may need to be replaced following steam cleaning).
- Ensure that the framework is thoroughly dried after cleaning.
- Always ensure that any moving or adjustable parts are re-lubricated.

24.0 General Maintenance

General maintenance should be carried out by a competent, professional person who is well informed on how to use the equipment (see warranty - page 33) if there is no such person available or a more thorough check is needed, a service via an approved repairer (see opposite) should be booked. The person who carries out the maintenance check or service should always fill out the service history log record (page 32) in order to ensure the warranty remains valid.

As documented opposite, the degree of maintenance required is dependant on various factors relating to the use of the product. In addition to the specific functional adjustment warnings specified in this manual, it should be ensured that a thorough inspection of the following should be completed at no greater than 6 month intervals:

NOTE: Please see the Maintenance Table on the next page.

NOTE: Points listed here are generic across the Sunrise Medical range and do not apply to all products.

Maintenance Table

Castors	Lift base and check each wheel to ensure it is not damaged, loose or worn.
Brakes	Removing any grease or dirt that has built up on the wheels, check that the brakes stop the wheels rotating. Check they are securely fixed in place & that there are no signs of damage or wear & tear.
Pushing Handles	Ensure handles are functioning correctly. Check there are no signs of damage or wear and tear. Ensure push handle grips are secure.
Seat Tilt In Space Mechanism	With the tilt lock released check for free movement with no signs of damage or wear to the mechanism, lever or cables. Tighten the lock and ensure it holds the seat angle.
Base Height Adjustment	With the seat unloaded, press the lever & ensure it moves freely to the maximum height. In incremental stages add weight up to the maximum load (40, 60, 80 or 100kgs) and, over a period of time (approx 10 - 15 minutes), check that there is no creep (lowering) of the seat. If it is not practical to perform this test, as a minimum requirement, immediately following the maintenance check, the static position of the seat should be monitored whilst a user is positioned in the system. With the load on, press the lever & ensure the seat travels freely to the lowest position.
Seat / Backrest Upholstery	Check the attachment points of the seat and backrest upholstery are tight. Check for worn covering. Rough worn upholstery can cause discomfort, would be difficult to keep hygienically clean, and may cause skin irritation.
Interface	Check there are no signs of damage or wear and tear. Ensure the function secures the seating unit properly.
Frame With correct maintenance the frame should provide at least 5 years trouble free use but, depending on conditions of use, wear will occur and a thorough inspection is recommended.	Check all metal parts to ensure there are no signs of damage or wear and tear, paying particular attention to adjustable or moving parts. Check there are no signs of failure in joints and welds. Check tightness & security of all fixings, bolts, nuts, spring loaded pegs, & other fitments. Check for signs of fatigue wears, replace parts that show signs of wear or repeatedly becoming loose. ANCHOR POINTS (where applicable) for transportation must be checked to ensure security, check there are no signs of damage, wear and tear or failure.
Fixtures	Check arm pads and headrest and replace if worn.

What should be completed during a service?

A service is a comprehensive combination of inspection, maintenance and repair or replacement of worn, faulty or missing components. The growth and any changes of the occupants needs since the original assessment is taken into account when performing the service. This is in contrast to general maintenance (opposite), where only straightforward checks, inspections and adjustments take place and primary services are planned.

What happens if a fault is found?

If any faults are found that could prove to be a risk to either the user or the operator then all use of the equipment should be ceased immediately until the product has been repaired.

Who should carry out the service?

It is stressed that only a Sunrise Medical approved repairer or a person with competent training of a Class 1 medical device should carry out this work.

Any modifications must not be carried out without prior agreement of Sunrise Medical Seating Solutions Ltd. It must be understood that unauthorised modifications may pose a risk to users and attendants. Unauthorised modifications will also invalidate the warranty.

Any new parts required should be genuine Sunrise Medical Seating Solutions Ltd. approved parts, fitted to Sunrise Medical specifications.

Frequency of Service

Sunrise Medical recommend that a service of this product is completed once every 6 months as a minimum. However, the frequency of inspection and service must be altered depending upon the severity of use. If the system is used in any of the following ways then the use could be said to be fairly heavy and constant. Therefore we would recommend the service interval be reduced to once every three months;

- Daily for around 8 to 10 hours.
- Weekly for at least 6 days out of 7.
- Monthly for at least 11 months a year.
- By an active user or somebody who is very active voluntarily or involuntarily.
- By a user who is above 80% of the maximum user weight recommended.
- Transported in a moving vehicle twice or more in a day.

CAUTION!

If the usage exceeds the amount highlighted above it is possible that services every six weeks or less might be needed.

How To Book A Service

Sunrise Medical have specified technicians trained to service our products. If you would like Sunrise Medical to service your chair or for further information please contact us on 01733 405830.

Alternatively, contact the person who issued you with the product.

26.0 Service Record Log

This log is an important record of usage and care of the product. It is a primary tool in predicting the need for further service and maintenance. It should be completed each time an inspection, service or any other significant manipulation has taken place. Sunrise Medical Seating Solutions Ltd. will require proof of service for any warranty claims or orders.

Seating system serial number _____
Date delivered _____ / _____ / _____
Battery use by date _____ / _____ / _____

Date	Serviced Performed	Organisation	Print Name & Sign

Please contact Sunrise Medical if additional log sheets are required.

28.0 Inspecting and Reissuing of Equipment

Most Sunrise Medical Seating Solutions Ltd. products are individually assessed for a client prior to issue. They are handed over in a controlled way to ensure optimum fit and specification, checks are completed and any additional individual verbal instructions given. We therefore recommend the following points are adhered to prior to any re-issue of equipment:

Cleaning

Follow a rigorous process of cleaning in order to decontaminate the product and eliminate the chance of cross infection.

Assessment of Fabrics / Foams

Carefully assess the condition of fabrics and foams in the seating product, and have these replaced partially or even completely if need be. In severe cases if the chassis is worn and the seating in poor condition it may be necessary to have the product re-furnished by the manufacturer. In very severe cases if the chassis is badly worn and the seating in very poor condition then the product should be decommissioned and disposed of.

Electrical and Electronic Equipment Decommissioning

If your product is set up with an electrical function you should always contact Sunrise Medical or your authorised representative for de-commissioning information.

Appropriateness of Equipment

Check that the equipment supplied is appropriate for the needs of the user taking age, weight, ability, diagnosis, and any other important factors into account. For instance, Sunrise Medical Seating Solutions Ltd. products may not be suitable for persons with severe challenging patterns of behaviour. Also ensure safe specified limits for use are not exceeded by the new user. Your sales representative can advise on the suitability of the equipment.

Manuals and Records

Ensure this instruction manual and any additional manuals for the seating unit are handed over to the new user with the equipment and ensure the servicing log history is up to date. Records on the new user must be updated to maintain traceability.

Inspection

The equipment should have a rigorous inspection prior to the reissue, to include the points covered in the six monthly routine check, regardless of if the checks were recently completed. The equipment should be free of excessive wear or distortion in any aspect of its main construction or accessories.

Condition

Wheels, tyres and brakes must be in good working condition, properly adjusted and inflated as appropriate.

Accessories

Check all accessories carefully for damage and potential shortcomings which may pose a risk to the user.

Function

Ensure that all the functions of the chair are working correctly e.g. tilt in space, height adjustment, back recline, folding etc.

Fixings

Check that any knobs, hand wheels, nuts, bolts, levers and fixings are in good condition and fitted in place securely.

Modifications

Any special modifications, adaptations, alterations or other such procedures including added accessories may require review. If they are found inappropriate, remedial action (if considered safe and practical) should be taken to reverse them if possible.

Labelling

Any labelling specific to the previous user should be removed and replaced with labelling specific to the new user.

Adjustment

Adjustment of the equipment to the new user must be carried out by suitably qualified and experienced professionals. These adjustments along with the other appropriate checks and tests should be completed with the owners knowledge and agreement.

Hand Over

Competent handing over of the equipment to the new user or attendant must include proper training and advice in safe use, particularly regarding transportation issues.

Packaging

Always ensure that the chair is packaged correctly before delivery.

Handle Fitting

Please note that the handle fitted is dependent on the size of the chair. You therefore need to ensure the size label on the handle matches the size of the chair being used.

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